# beurer

## HealthManager Pro

<b>DE</b> beurer HealthManager Pro Gebrauchsanweisung .....	<b>HU</b> beurer HealthManager Pro Használati útmutató .....
2	122
<b>EN</b> beurer HealthManager Pro Instructions for use.....	<b>RO</b> beurer HealthManager Pro Instrucțiuni de utilizare.....
10	130
<b>FR</b> beurer HealthManager Pro Mode d'emploi.....	<b>EL</b> beurer HealthManager Pro Οδηγίες χρήσης .....
18	138
<b>IT</b> beurer HealthManager Pro Istruzioni per l'uso .....	<b>SL</b> beurer HealthManager Pro Navodila za uporabo .....
26	146
<b>ES</b> beurer HealthManager Pro Instrucciones de uso .....	<b>PT</b> beurer HealthManager Pro Instruções de uso .....
34	154
<b>TR</b> beurer HealthManager Pro Kullanım kılavuzu.....	<b>BG</b> beurer HealthManager Pro Инструкции за употреба .....
42	162
<b>RU</b> beurer HealthManager Pro Инструкция по применению .....	<b>HR</b> beurer HealthManager Pro Uputa za uporabu .....
50	170
<b>PL</b> beurer HealthManager Pro Instrukcja obsługi .....	<b>LT</b> „beurer HealthManager Pro“ Naudojimo instrukcija .....
58	178
<b>NL</b> beurer HealthManager Pro Gebruiksaanwijzing .....	<b>LV</b> beurer HealthManager Pro Lietošanas pamācība .....
66	186
<b>DA</b> beurer HealthManager Pro Betjeningsvejledning.....	<b>TH</b> beurer HealthManager Pro คู่มือการใช้งาน.....
74	194
<b>SV</b> beurer HealthManager Pro Bruksanvisning.....	<b>KO</b> beurer HealthManager Pro 사용 설명서 .....
82	202
<b>NO</b> beurer HealthManager Pro Bruksanvisning.....	<b>ID</b> beurer HealthManager Pro Petunjuk penggunaan .....
90	210
<b>FI</b> beurer HealthManager Pro Käyttöohje.....	<b>ZHCN</b> beurer HealthManager Pro 使用说明 .....
98	218
<b>CS</b> beurer HealthManager Pro Návod k použití .....	<b>ZHTW</b> beurer HealthManager Pro 使用說明書 .....
106	226
<b>SK</b> beurer HealthManager Pro Návod na použitie .....	
114	

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Read these instructions for use carefully. Keep these instructions for use for future reference. Be sure to make them accessible to other users and take note of the information they contain.

**Table of contents**

- 1. Signs and symbols ..... 10
- 2. Intended use..... 11
  - 2.1 beurer HealthManager Pro ..... 11
  - 2.2 Mean arterial blood pressure calculation module..... 11
- 3. Warnings and safety notes ..... 11
- 4. App description ..... 12
  - 4.1 Installation ..... 12
  - 4.2 Dashboard ..... 12
  - 4.3 Menu ..... 12
  - 4.4 Add device ..... 13
  - 4.5 Add manual measurement ..... 13
  - 4.6 Measurement detail page..... 13
  - 4.7 History ..... 14
  - 4.8 Goal ..... 14
  - 4.9 Device settings ..... 15
  - 4.10 Export function ..... 15
  - 4.11 Medication..... 15
  - 4.12 Comment function..... 16
- 5. What if there are problems? ..... 16
- 6. Uninstalling the app..... 16
- 7. Compatibility ..... 17

**1. SIGNS AND SYMBOLS**

Symbols used in the app are explained there by clicking on the respective symbol. The following symbols are used in the instructions for use:

	<b>Warning</b> Warning notice indicating a risk of injury or damage to health
	<b>Important</b> Safety note indicating possible damage to the device/accessory
	<b>Product information</b> Note on important information
	Consult instructions for use
	Manufacturer
	<b>CE labelling</b> This product satisfies the requirements of the applicable European and national directives
	Medical device
	Unique Device Identifier (UDI) for unique product identification

<b>REF</b>	Item number
<b>LOT</b>	Batch code
<b>#</b>	Type number

## 2. INTENDED USE

### 2.1 beurer HealthManager Pro

#### Intended Use

The “beurer HealthManager Pro” is a system for digitally monitoring your health. Using the “beurer HealthManager Pro“, health-related data from measuring devices such as blood pressure or blood glucose monitors can be stored, archived, managed and exported, for example. Progress graphs, tables and individual comments provide you with a quick and convenient overview of all your values. Device parameters can also be set using the application. The application is not a system for diagnosing illnesses and does not make any therapeutic suggestions.

#### Target group

The target group is users who want to transfer health-related data from home medical devices to electronic devices such as mobile phones, tablets or home computers using an application.

### 2.2 Mean arterial blood pressure calculation module

#### Purpose

The module determines the mean arterial blood pressure from the systolic and diastolic blood pressure values.

#### Indication

The calculated mean arterial blood pressure value is used to provide information in the event of disorders such as hypertension. This can also be used on healthy people to prevent blood pressure disorders.

#### Clinical benefits

For monitoring and preventing blood pressure disorders.

#### Contraindication

People with impaired physical, sensory or mental capabilities should be supervised by a person responsible for their safety.

## 3. WARNINGS AND SAFETY NOTES

### Recommended updates

To ensure safe and optimal use of the “beurer HealthManager Pro” app, it is advisable to install software updates as soon as they are available.

### General warnings

- The measurements taken by you are for your information only – they are no substitute for a medical examination! Discuss the measured values with your doctor and never make your own medical decisions based on them (e.g. regarding dosages of medicines).
- The app and the module are only intended for the purpose described in these instructions for use. The manufacturer is not liable for damage resulting from improper or incorrect use.
- The underlying health data is sensitive data and should therefore be treated confidentially.

## 4. APP DESCRIPTION

### 4.1 Installation

1. Open the Apple App Store (iOS) or Google Play Store (Android) on your smartphone.
2. Search for the „beurer HealthManager Pro“ app\*.
3. Follow the instructions and download the app. The installation starts automatically.
4. As soon as the installation process is complete, you can launch the app.

Click here for the beurer HealthManager Pro app \*

\* This product satisfies the requirements of the applicable European directives.

### 4.2 Dashboard

The dashboard is the app's home page. Here the user can find a clear display of their active categories and connected devices. Clicking on the category tile takes the user to the measurement detail page of the respective category. Using the device tile, the user can access the settings area of the connected device.

You can also access the menu from the dashboard by clicking on the three dashes at the top left. The goal settings and the history are called up by clicking on their symbol tiles. New devices and manual measurements can be added using the + in the top right corner.


### 4.3 Menu

The menu is accessed from the dashboard by clicking on the three dashes at the top left.

The menu can be closed by clicking on the dashboard area on the right-hand side of the screen.

The following areas of the app can be accessed from here: Settings, Services, Help and Legal information. It is also possible to switch between different profiles if the multi-user function is used.

**Note:** The multi-user function is only possible for users with an online account.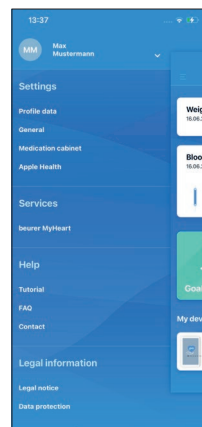


## 4.4 Add device

The + symbol at the top right of the dashboard takes the user to the “Add device” area.

Here they can select the category from which they want to add a device. After clicking on the relevant category, the selection of applicable devices opens. The user must now select the device they want to add from here. Devices that are already connected are greyed out here and cannot be reselected.

The guided integration of the respective device starts from here.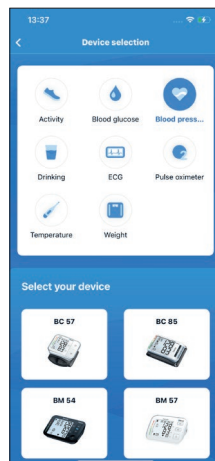


## 4.5 Add manual measurement

Manual measurements can be recorded in each category using the Add a measurement function.

Here, the time and the respective measurement are recorded per category. A comment and medication can also be entered here directly.


Clicking the Save button saves the entire entry and adds it to the respective category.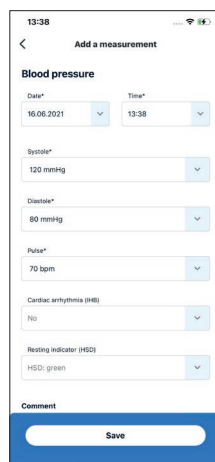

## 4.6 Measurement detail page

The current measurement is displayed prominently on each measurement detail page. This measurement is evaluated according to either internationally accepted guidelines or personal goals. This evaluation is represented by the coloured circle around the measurement. Clicking on the measurement circle opens information regarding the reference values or your personal goal. Optionally, relevant information regarding the measurement is also shown. The user can also click on this to read about the relevant meaning.


The user can navigate to another measurement in the same category by swiping through the date and time selection at the top of the screen. Measurements can be edited, added and deleted using the more symbol (three dots in the top right corner). Comments and medication can be added to the respective measurement. In addition, the selected measurement can be shared. In the blood pressure category, a guided triple measurement can also be started via the more symbol. The goal tile is used in each category to clearly display the individually set goals if they have been activated. Some categories require an individual goal. This cannot then be deactivated.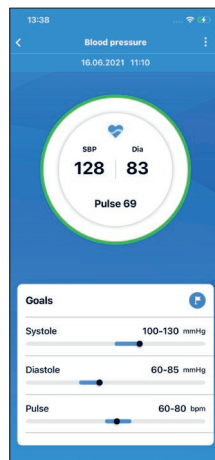

The course of measurement is located on each measurement detail page. All previous measurements are displayed graphically here. Different settings can be configured for each category, e.g. the time period can be selected. By clicking on the individual measuring points, the relevant information on the respective measurement appears.

Swiping to the side allows the user to navigate through the graph in terms of time. Clicking on the calendar symbol at the bottom left of the course of measurement opens up the calendar. A specific time can now be selected here if it contains measurements.

By clicking on “Show all measurements”, the user can access the history, which lists all measurements filtered according to the respective category.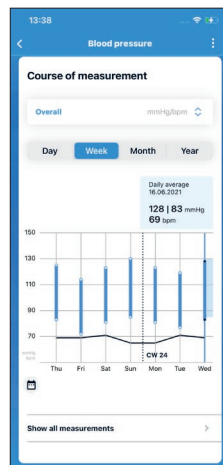

## 4.7 History


All previously recorded measurements, comments and medications are listed chronologically in the history.

A specific time can be selected using the calendar symbol at the top left.

Certain categories can be selected or deselected using the filter symbol at the top right.


Measurements, comments and medication can be added using the more symbol (three dots at the top right). In addition, measurements, comments and medication can be deleted via “Edit history”.

All existing values can be exported via the export function. You can select the desired time period and categories and choose between PDF and CSV data formats. The export report is then created and can be sent via the Share function of the smartphone operating system, e.g. as an e-mail or message.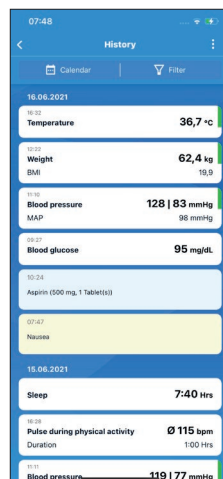


## 4.8 Goal

In the goal area of the app, an individual goal can be selected for each category. Reference values are stored here by default, but these can be adapted to your own requirements. The set goals can also be adapted or deactivated here if they are not required for the correct display of the category.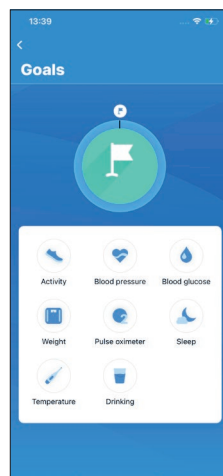


## 4.9 Device settings

In the device settings, specific information can be displayed for each connected device and settings configured.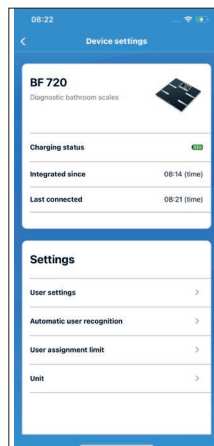


## 4.10 Export function

All existing values can be exported via the export function. You can select the desired time period and categories and choose between PDF and CSV data formats. The export report is then created and can be sent via the Share function of the smartphone operating system, e.g. as an e-mail or message.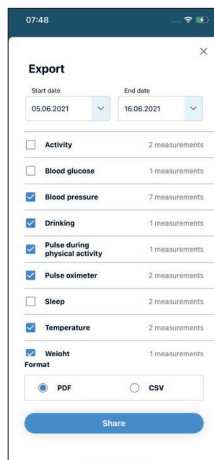


## 4.11 Medication

The medication function can be used to create a medication entry and document your intake of it. Medication entries can be created and saved in conjunction with measurements or independently.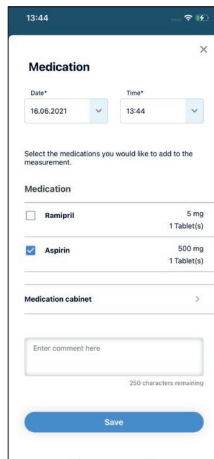

## 4.12 Comment function

The comment function allows free-text comments to be recorded separately or in conjunction with a specific measurement.

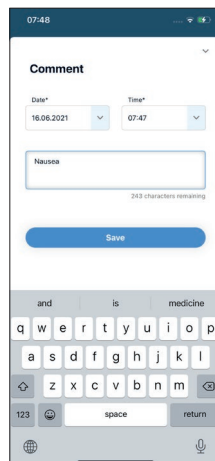### Notes on registration:

The app can be used both locally and with an online account. Depending on the specific product/service, it may not be essential to register.

An online account must be created in order to use the cloud function for data backups.

### Notes on data logging:

Whilst you are using it, the app generates a log file containing technical information about the communication between the app and device as well as profile information. It is saved locally on the smartphone and, in the event of a support enquiry, can optionally be sent to the Customer Service team via the contact form.

## 5. WHAT IF THERE ARE PROBLEMS?

Problem	Remedy
My app stops responding or crashes.	Completely close and restart the app.
The app is displayed incorrectly on my smartphone.	Check whether your smartphone is on the list of compatible devices. If this is not the case, unfortunately we cannot guarantee that the app will work correctly on your smartphone.
Not all measurements are shown in the history.	Check whether the filter function is activated using the filter symbol at the top right.
I have a problem that cannot be solved using the instructions.	Please contact our customer service via Menu → Help → Contact and describe your problem as accurately as possible.

## 6. UNINSTALLING THE APP

### iOS:

1. Hold down the “HealthManager Pro” app icon with your finger until it starts to move.
2. Tap the small “-” that appears in the top corner of the screen.
3. A question will appear on the screen asking if you want to delete the app. Tap “Delete” here. The app is now uninstalled.

## Android:

1. Open the “Settings” of your Android smartphone/tablet.
2. Tap on “Apps”
3. Tap on the “HealthManager Pro” app.
4. Tap “Uninstall”. After confirmation, the app is deleted.

## 7. COMPATIBILITY

List of system requirements and compatible devices:

### Notification of incidents

If during or through use of the product a major incident occurs, notify the manufacturer and/or their representative of this as well as the respective national authority of the member state in which the user/patient is located.

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
Klikom na gumb Spremi sprema se cijeli unos i dodaje u odgovarajuću kategoriju.

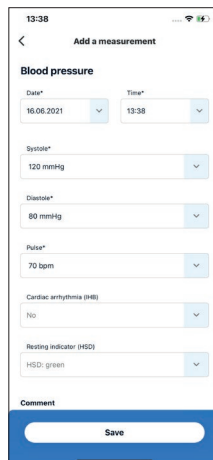The screenshot shows the 'Add a measurement' screen for blood pressure. It features a title bar with the time 13:38 and a back arrow. Below the title is the text 'Add a measurement' and 'Blood pressure'. The form includes fields for 'Date\*' (16.06.2021), 'Time\*' (13:38), 'Systole\*' (120 mmHg), 'Diastole\*' (80 mmHg), 'Pulse\*' (70 bpm), 'Cardiac arrhythmia (HR)' (No), and 'Rating indicator (HSD)' (HSD: green). There is a 'Comment' field at the bottom and a 'Save' button.

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Apple and the Apple logo are trademarks of Apple Inc., registered in the U.S. and other countries.
Google Play and the Google Play logo are trademarks of Google LLC.
Android is a trademark of Google LLC.
App Store is a service mark of Apple Inc., registered in the U.S. and other countries.